


Ear Notch PI BVD Sampling

1. Sequentially label the sample vials with the ear tag ID using a permanent marker.
2. Prepare 3 buckets for cleaning the ear notching tool. Fill 2 buckets with tap water and the third with a bleach water mix (10% bleach w/ the rest water).
3. Using an approved ear notching tool, take a full sample from the upper corner of the outside of the ear.
4. Place the notch in the vial.
5. Clean ear notching tool after each use. First rinse in tap water. Second wash is bleach water. Third rinse thoroughly in second bucket of tap water.
6. If shipping within 48 hours of collecting, FREEZE samples until shipping.
7. Completely fill out BVD submission form.
8. When shipping, please pack on ice.
9. Please include payment for samples at time of submission. The test cost is \$3.50/sample.
10. Results can be sent via mail, fax, email, or phone. Please specify your preference when submitting samples.
11. Ship samples to:

SEK Genetics
9525 70th Road
Galesburg, KS 66740
1-800-443-6389.

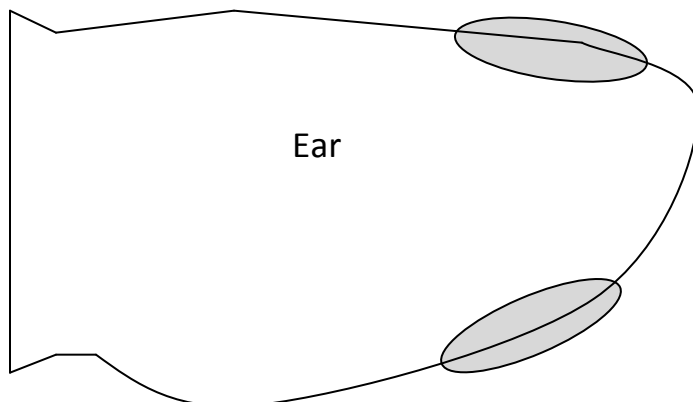

Take sample from one of the shaded areas.